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Clicca **Create an account**

### What is ns.10be.de?

Well, for your own **Nightscout** To set up an instance, some work is already near. So I automated it a bit further. So you can now easily over the **profile** (after a few instances).

In short, it's a **Managed Nightscout Service**. Apart from your settings (with a cli) The rest is done automatically by **10BE** in the background for the user.

The exact process in the background is explained below **Features**

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I accept the **Conditions and Terms and Privacy Policy** (how long we save the reports, storage time).

AND: I am aware that this is an Experimental and NONmedical service that in no way replaces the medical therapy given by my doctor and should not be used for this purpose. Therefore, I exempt the operator, agents, etc. from any responsibility regarding the use of this service.

Yes, I understand that the system is NOT designed for diagnosis or screening.

I am aware that the system is purely information management software (diary/data storage) and is not a substitute for the advice of medical professionals.

If you have any questions about therapy, it is essential to contact the specialist staff!

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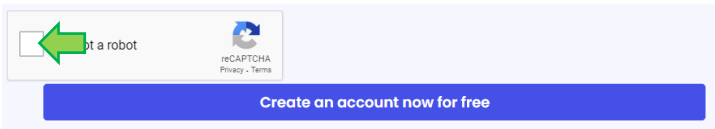
## Register a Account for ww.ns.10be.de

**Thank you for your Registration.**

We have just send a message with a link to the address you specified: \_ @ .

Please confirm the link within 7 days, as otherwise the link will become invalid. (During the period no further registration with the registered data are possible only after the end of the period, confirmation or rejection of registration).

Under </en/register.resend.html>, the registration mail can be sent again in a few minutes if the mail has not arrived.

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Link confirmation of registration for the account on ns.10be.de Posta in arrivo x

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Hello glicemiadistanza

thank you for your registration on <https://ns.10be.de/>

So that the registry is finally completed is a confirmation of the holder of the given e-mail address ( ) is necessary.

If you want to confirm the registration, please click on the link below:  
<https://ns.10be.de/en/register.confirm.html?regkey=50062932-1008525914>

~~If you have not registered with us, so you ignore the e-mail, because all unconfirmed registrations expire automatically after 7 days.  
You can lock your address for other entries by permanently following link:  
<https://ns.10be.de/en/register.disable.html?regkey=50062932-1008525914>~~

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## Registration Step 2 from 2

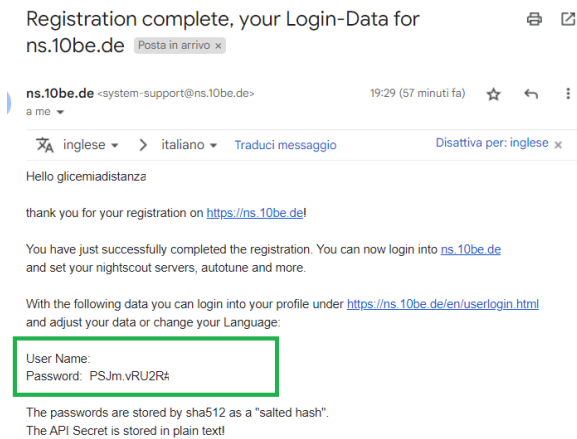
**Thank you to confirm your Registration.**

Registration is now complete.


You will receive an additional e-mail from us with your account data. These, and any other information you can always edit your profile.

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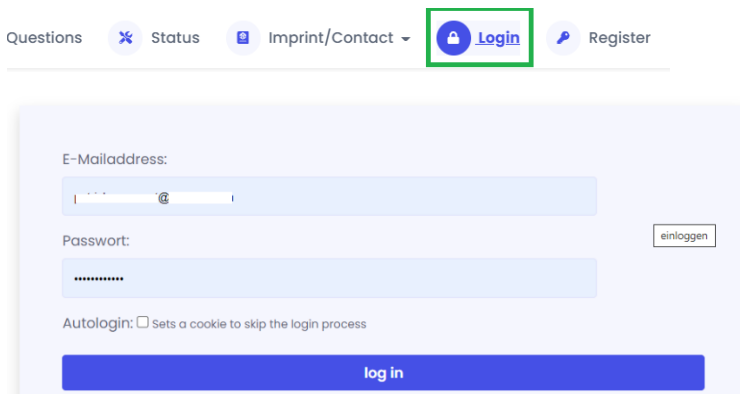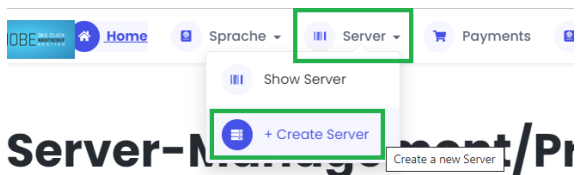
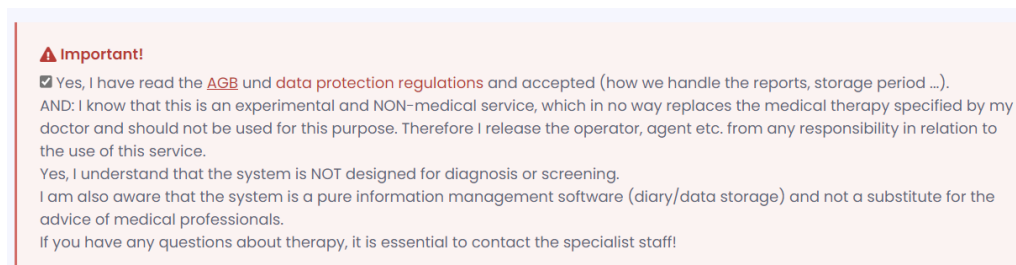
**Name:**  
Is used for the public nightscout URL (xxxxname.ns.10be.de)  
Numbers, letters and - only!  
.ns.10be.de

**Display-Name:**  
can remain empty

**API-Password:**  
The password can be chosen freely (at least 12 characters long)  
(but stored in plain text in the database).  
Special characters like !#%&/= Have to be escaped in xDrip.

Se usi Spike o Loop, abilita **Without port**.

**Without-Port:**  
If activated, the server is also ONLY accessible with https:// WITHOUT the Port. Iphone loop user, have to activate this, because loop ignores the port!  
 Activate to call up the page directly via https://xxx.ns.10be.de/

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**Serverlocation:**  
If activated, it will use cluster and MongoDB server from Location Germany.  
 Serverlocation Germany

**Branch:**

**12/24h Hours:**  
 24h  12h 

**Alert for High:**  activate  
BG\_HIGH:   
BG\_TARGET\_TOP:

**Alert for Low:**  activate  
BG\_LOW:   
BG\_TARGET\_BOTTOM:

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▼ LibreLinkUP (LibreLinkUP-Follower, fsI3)

LibreLinkUP (Follower) librelink\_user\_name More Infos

LibreLinkUP (Follower) librelink\_password More Infos

**will be saved in plain text!**

LibreView librelink\_server More Infos  
 LibreView librelink\_interval (1 Min default) Weitere Infos

Every 5 minutes instead of every minute?

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**STATUS:** creating  
**IMPORTANT!**  
There is no payment for this Server. Please pay first under [Payments \(new tab\)](#).

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**STATUS:** creating  
**IMPORTANT!**  
There is no payment for this Server. Please pay first unde [Payments \(new tab\)](#).

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Or:

- 14,99 Euro for 1 **DB Export** (no subscription possible)

**Server:**  
If "empty" the payment is assigned to a server during the action (start, redeploy, create...).

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**With Subscription** with automatic payment and extension from the selected term:

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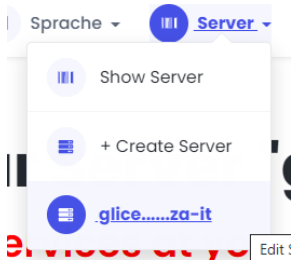
**Without Subscription**, always has to be paid manually here:

**stripe.com:** (Creditcard, SEPA, Google Pay, Transfer...) 

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
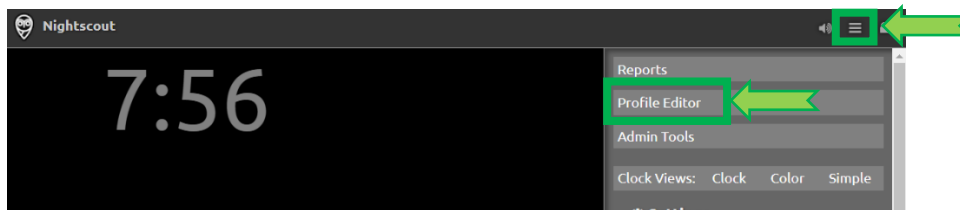
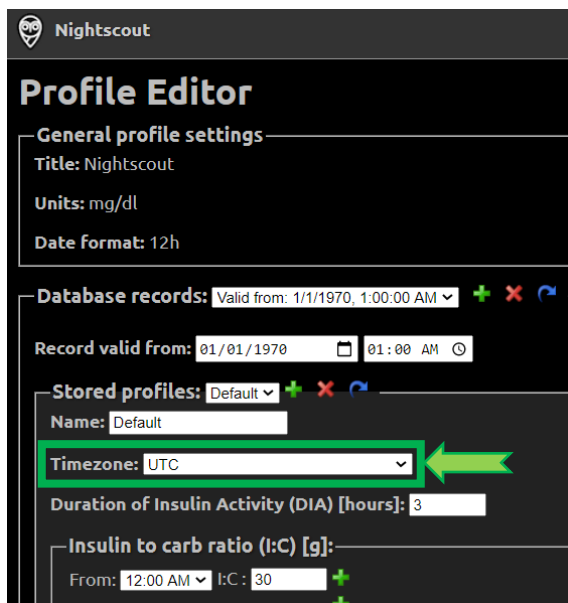
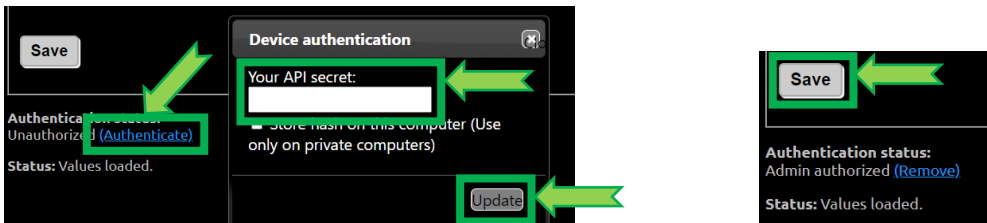
mynewcgm.herokuapp.com says

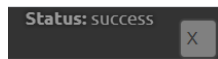Profile is going to be saved in newer format used in Nightscout 0.9.0 and above and will not be usable in older versions anymore. Are you sure?

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