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

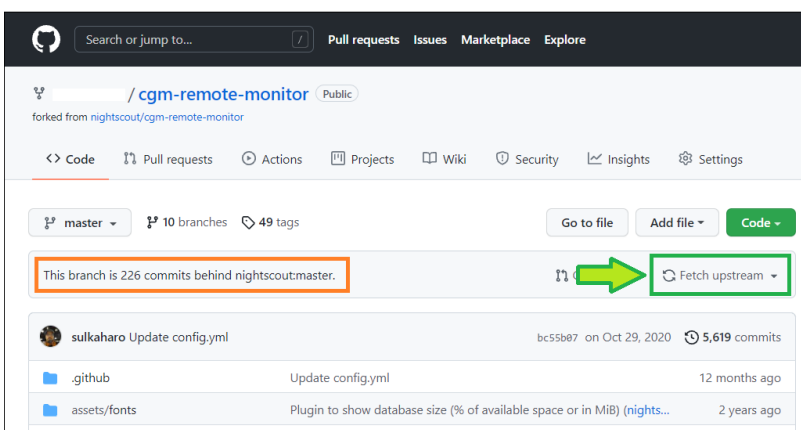
Clicca **Fetch and merge**.

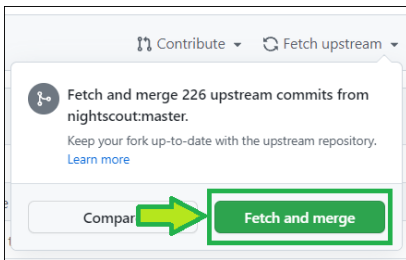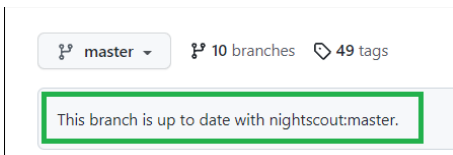
Dopo pochi istanti comparirà **This branch is up to date with nightscout:master**.

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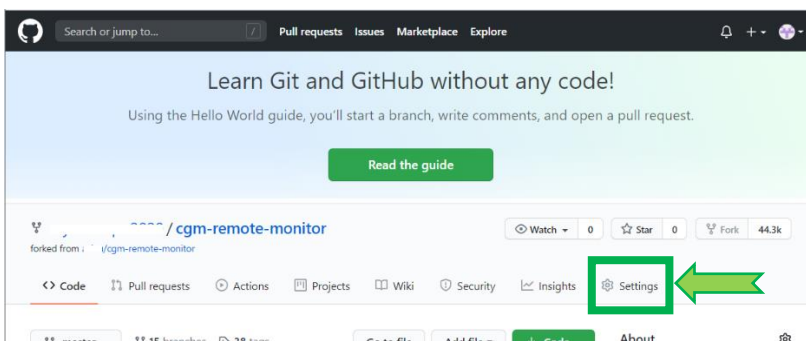Scorri giù a Danger Zone e clicca Delete this repository.

Danger Zone

Change repository visibility You cannot change the visibility of a fork. Please duplicate the repository .	Change visibility
Transfer ownership Transfer this repository to another user or to an organization where you have the ability to create repositories.	Transfer
Archive this repository Mark this repository as archived and read-only.	Archive this repository
Delete this repository Once you delete a repository, there is no going back. Please be certain.	Delete this repository

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Are you absolutely sure?

Unexpected bad things will happen if you don't read this!

This action **cannot** be undone. This will permanently delete the `nightscout/cgm-remote-monitor` repository, wiki, issues, comments, packages, secrets, workflow runs, and remove all collaborator associations.

Please type `nightscout/cgm-remote-monitor` to confirm.

I understand the consequences, delete this repository

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<https://github.com/nightscout/cgm-remote-monitor>

Quindi clicca su Fork

Search or jump to... Pull requests Issues Marketplace Explore

Learn Git and GitHub without any code!

Using the Hello World guide, you'll start a branch, write comments, and open a pull request.

Read the guide

nightscout / cgm-remote-monitor

Watch 248 Star 1.3k Fork

Code Issues 162 Pull requests 26 Actions Projects Wiki Security Insights

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Forking nightscout/cgm-remote-monitor

It should only take a few seconds.

Search or jump to... Pull requests Issues Marketplace Explore

Learn Git and GitHub without any code!

Using the Hello World guide, you'll start a branch, write comments, and open a pull request.

Read the guide

nightscout / cgm-remote-monitor

Watch 0 Star 0 Fork 44.3k

Code Pull requests Actions Projects Wiki Security Insights Settings

master 23 branches 42 tags

Go to file Add file Code

This branch is even with nightscout/master.

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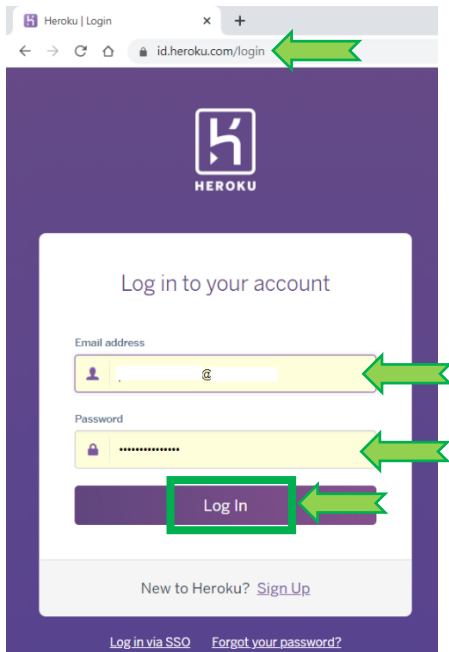
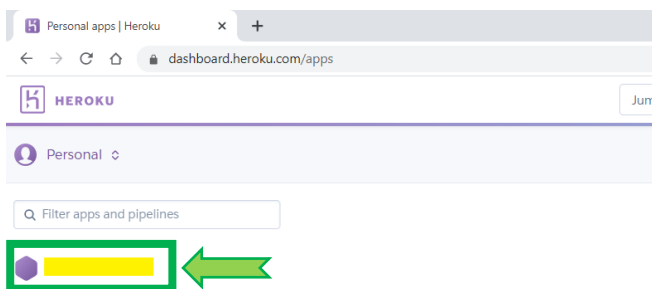
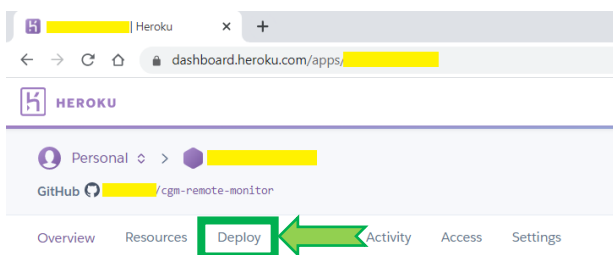
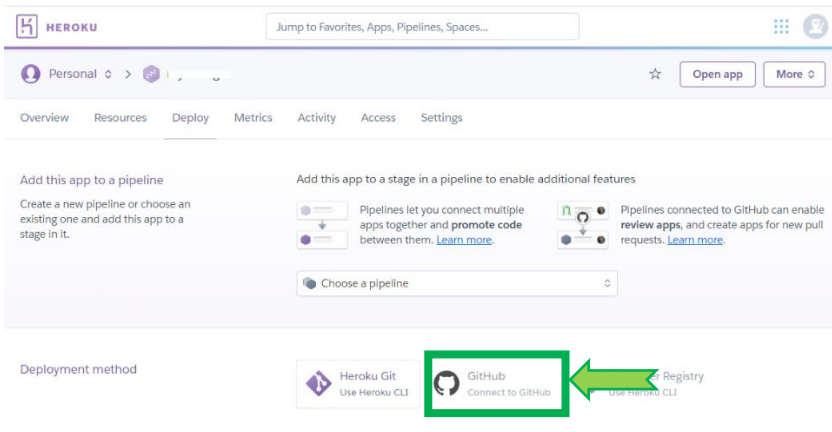
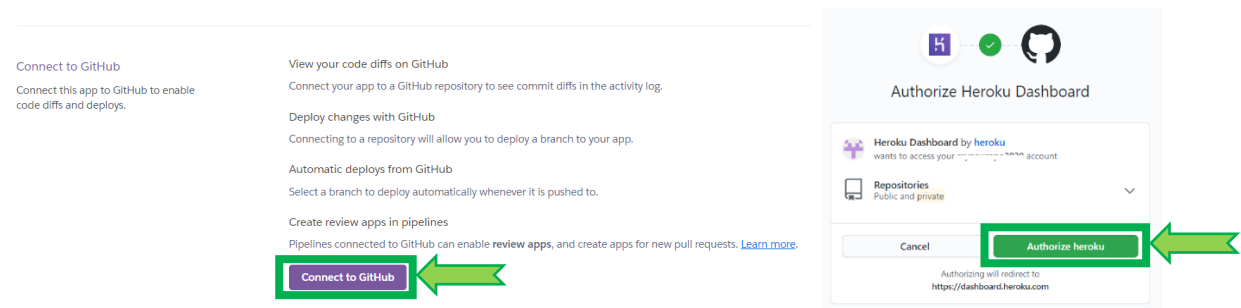
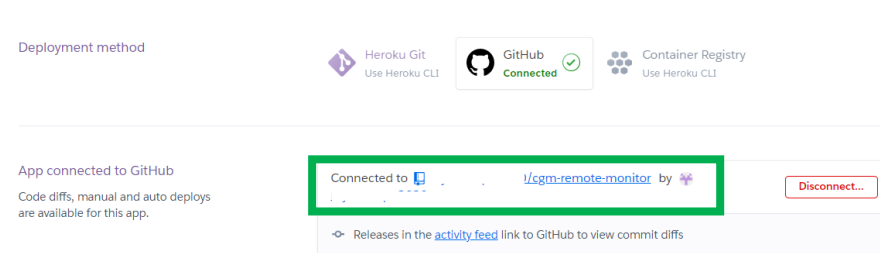
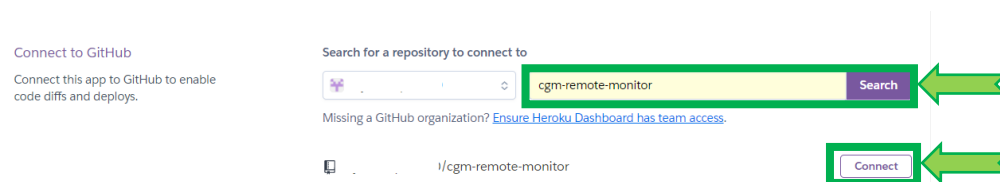
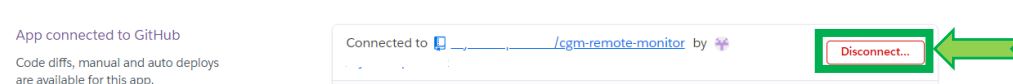
Manual deploy

Deploy the current state of a branch to this app.

Deploy a GitHub branch

This will deploy the current state of the branch you specify below. [Learn more.](#)

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Release phase

Deploy to Heroku

Your app was successfully deployed.

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