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Scorri giù a Danger Zone e clicca Delete this repository.

### Danger Zone

Change repository visibility  
You cannot change the visibility of a fork. Please [duplicate the repository](#). Change visibility

Transfer ownership  
Transfer this repository to another user or to an organization where you have the ability to create repositories. Transfer

Archive this repository  
Mark this repository as archived and read-only. Archive this repository

Delete this repository  
Once you delete a repository, there is no going back. Please be certain. Delete this repository

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Are you absolutely sure? ×

Unexpected bad things will happen if you don't read this!

This action **cannot** be undone. This will permanently delete the `nightscout/cgm-remote-monitor` repository, wiki, issues, comments, packages, secrets, workflow runs, and remove all collaborator associations.

Please type `nightscout/cgm-remote-monitor` to confirm.

`nightscout/cgm-remote-monitor`

I understand the consequences, delete this repository

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<https://github.com/nightscout/cgm-remote-monitor>

Quindi clicca su Fork

Search or jump to... 7 Pull requests Issues Marketplace Explore 🔔 + 🌐

Learn Git and GitHub without any code!  
Using the Hello World guide, you'll start a branch, write comments, and open a pull request.

Read the guide

`nightscout/cgm-remote-monitor` Watch 248 Star 1.3k Fork 👤

<> Code 🕒 Issues 162 🔗 Pull requests 26 🔄 Actions 📁 Projects 📖 Wiki 🔒 Security 📄 Insights

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### Forking nightscout/cgm-remote-monitor

It should only take a few seconds.

Refresh

Search or jump to... 7 Pull requests Issues Marketplace Explore 🔔 + 🌐

Learn Git and GitHub without any code!  
Using the Hello World guide, you'll start a branch, write comments, and open a pull request.

Read the guide

`nightscout/cgm-remote-monitor` Watch 0 Star 0 Fork 44.3k

<> Code 🔗 Pull requests 🔄 Actions 📁 Projects 📖 Wiki 🔒 Security 📄 Insights ⚙️ Settings

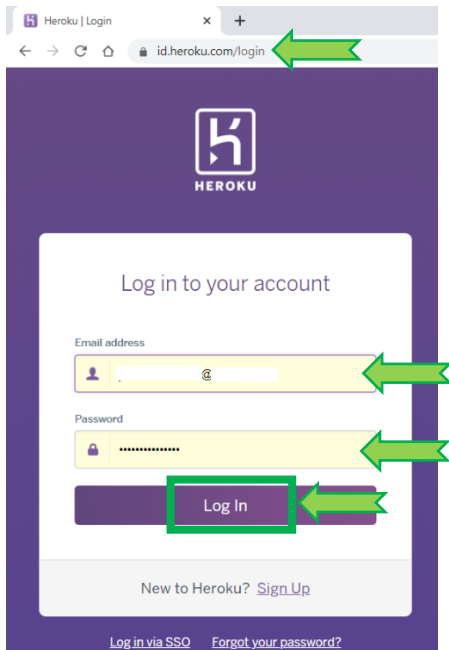
🔗 master 🌿 23 branches 🏷️ 42 tags Go to file Add file Code About

This branch is even with nightscout/master. 🔗 Pull request 🔗 Compare

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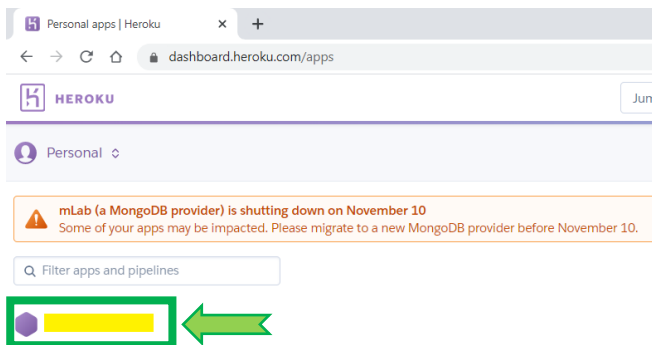
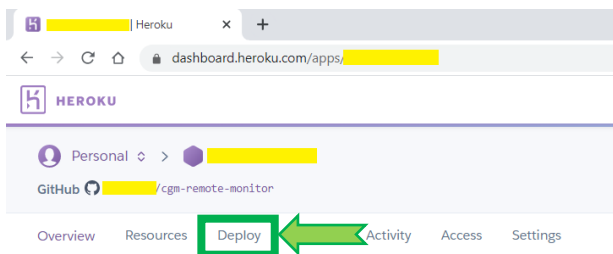
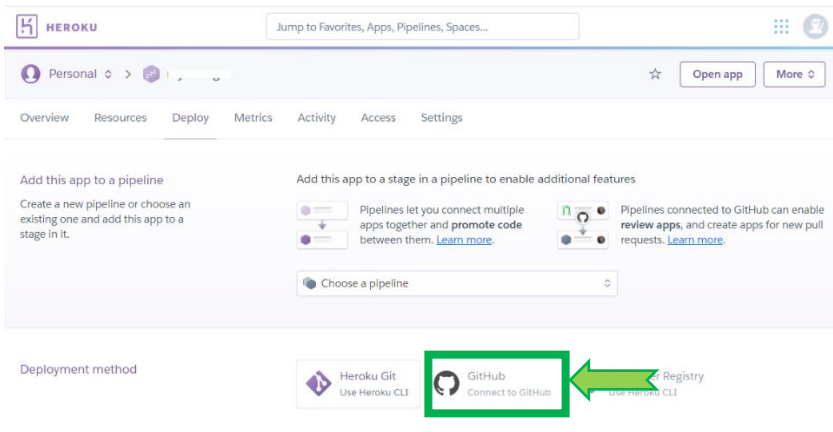
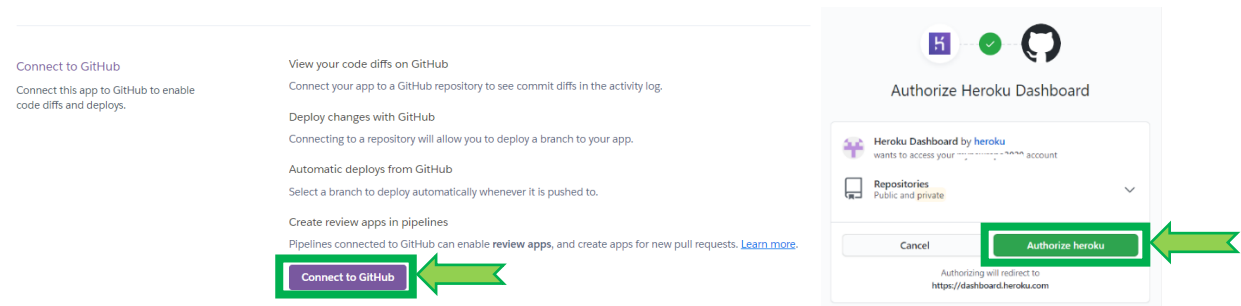
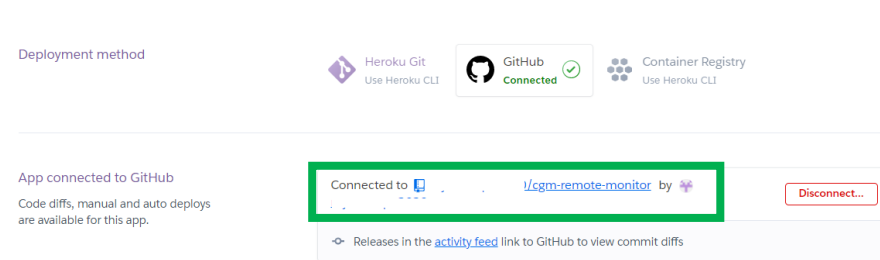
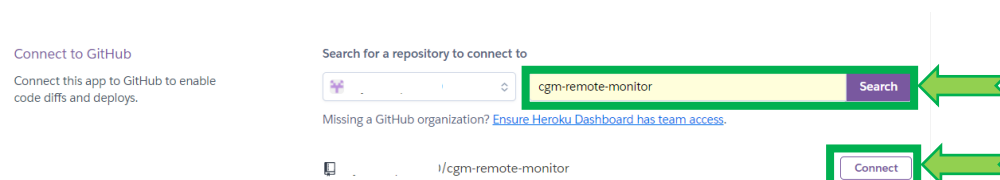
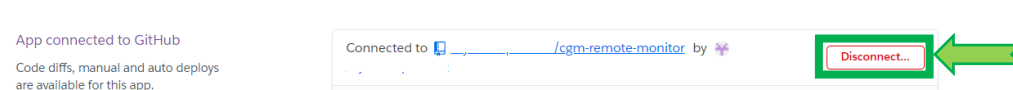
### Manual deploy

Deploy the current state of a branch to this app.

### Deploy a GitHub branch

This will deploy the current state of the branch you specify below. [Learn more.](#)

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Release phase

Deploy to Heroku

Your app was successfully deployed.

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