Monitor Nightscout Clock

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☰ README.md

If you already have an Ulanzi clock, you can try [installing](#) the Nightscout clock firmware

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Nightscout clock installer

This page is part of the [Nightscout Clock](#) project. It allows you to transform an Ulanzi TC001 programmable clock into a device showing blood sugar values through [Nightscout](#)

Here are the steps to install the firmware

- Buy Ulanzi TC001 from Aliexpress (e.g. from [here](#))
- Turn on the device by pressing < and > buttons for 3 seconds
- Connect the device to your computer using the USB cable
- Click the **CONNECT** button below
- Follow the instructions

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
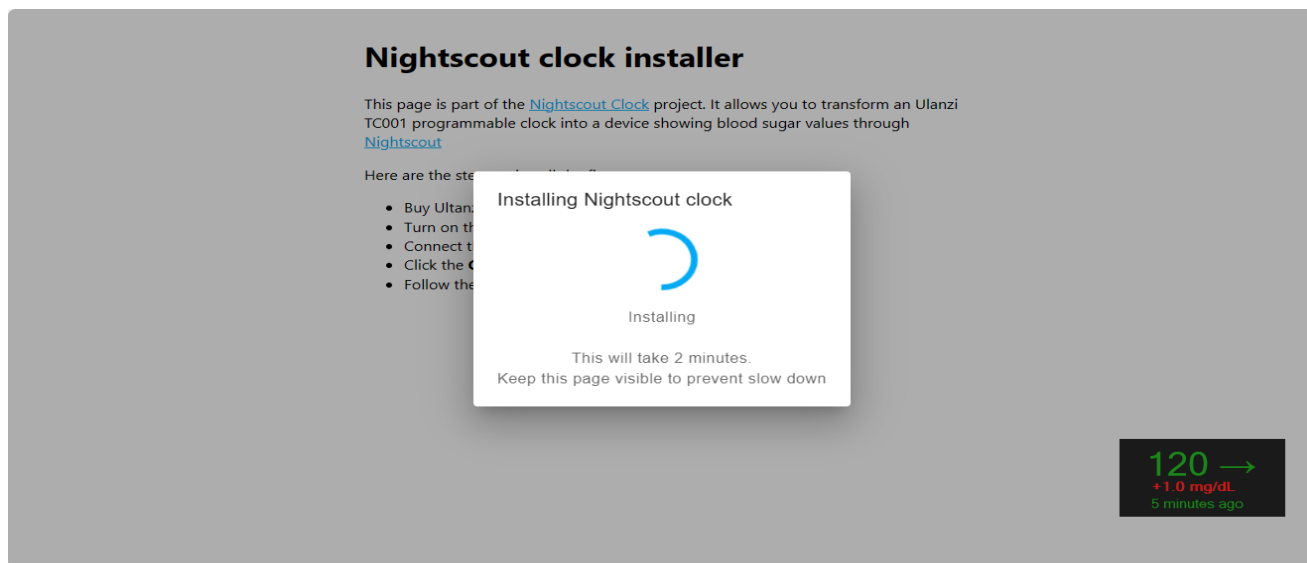
The screenshot shows the 'Nightscout clock' configuration page. It is divided into two main sections: 'Wireless network settings' and 'Nightscout settings'.
Under 'Wireless network settings', there are two input fields: 'Wireless network name (SSID)' and 'WiFi password'.
Under 'Nightscout settings', there are three input fields: 'Protocol' (set to HTTPS), 'Nightscout hostname (Hostname, not URL)' (set to glicemiad...), and 'Port (Optional)'.
Below these is an 'API secret' field with a note: '(Only needed when viewing Nightscout data is protected)'.
At the bottom, there are three input fields: 'Blood glucose units' (set to mg/dl), 'BG low value' (set to 70), and 'BG High value' (set to 180).
A blue button at the bottom right says 'Validate, save and restart'.
On the right side of the form, there are two glucose level indicators: '88 → -4.0 mg/dL 5 minutes ago' and '94 → -5.0 mg/dL 3 minutes ago'.

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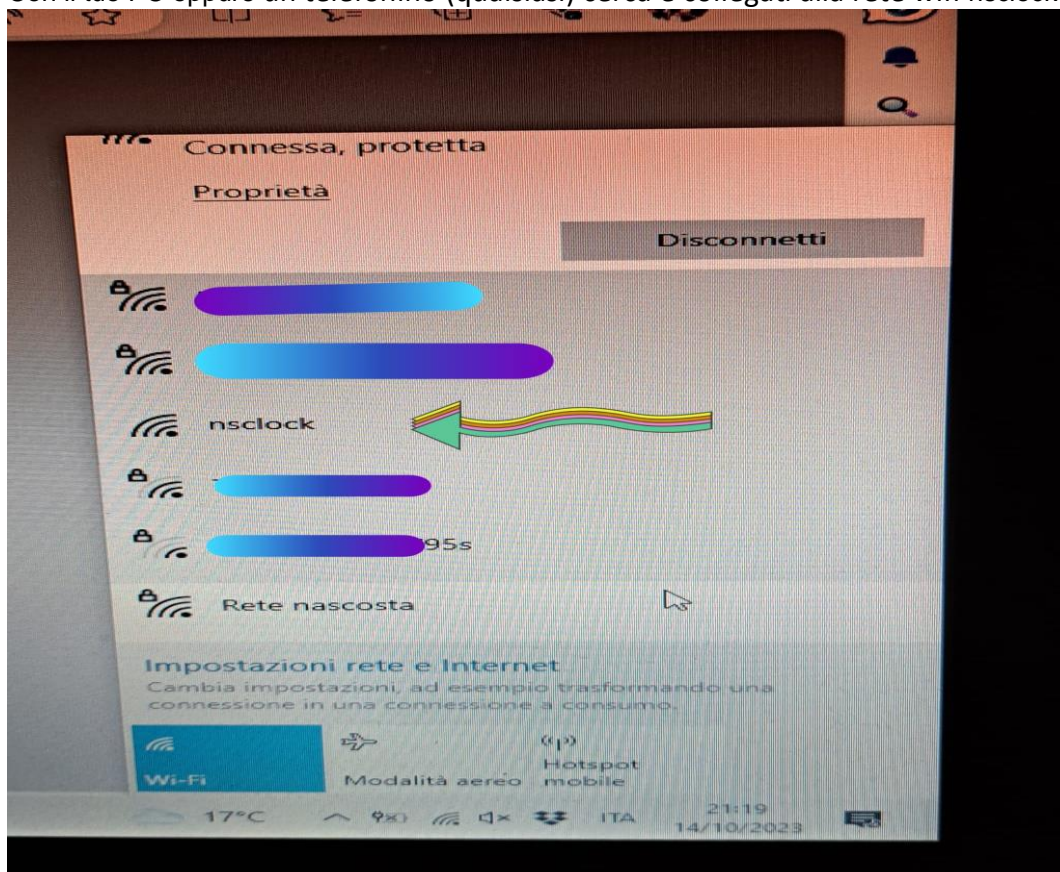Nightscout clock

Wireless network settings

Wireless network name (SSID) WiFi password

Nightscout settings

Protocol Nightscout hostname (Hostname, not URL) Port (Optional)

HTTPS glicemiad

API secret (Only needed when viewing Nightscout data is protected)

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